
PROFLEX® WRIST SUPPORTS


400/405

1. Wrap strap around entire wrist.
2. Secure in place with the hook-and-loop closure. Use on right or left hand.

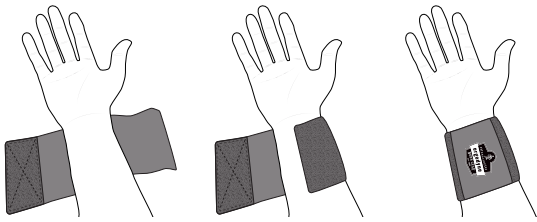
415

1. Feed strap through grommet.
2. Slip over hand to wrist just below palm.
3. Wrap strap around wrist, until the level of support feels comfortable. Use on right or left hand.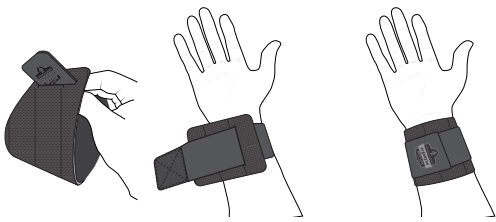


425

1. Insert thumb into hole.
2. Wrap support past palm, over the top of hand.
3. Loop around the wrist to secure.
4. Alternatively, wrap across the top of hand and then past palm before looping around the wrist to secure. Use on right or left hand.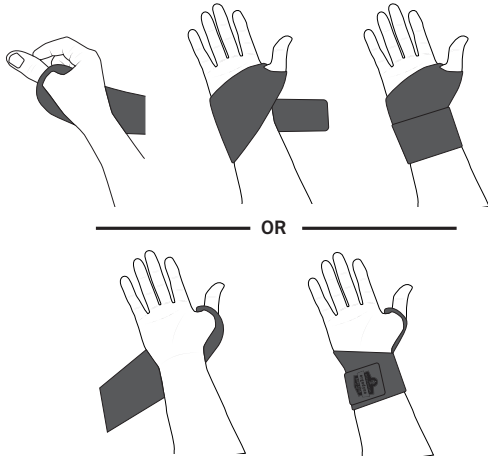


660

1. Insert hand into single opening end of support, feeding thumb through the small hole and remaining fingers through the larger opening.
2. Tug at base of wrist until material is even. Use on right or left hand.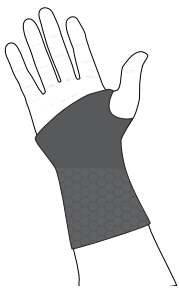


670/675

1. Slip hand and thumb through opening.
2. Place removable strap with loop to hook so the strap lays away from the body.
3. Pull strap underneath to wrist side and then over the back of the wrist.
4. Fasten hook-and-loop closure(s) to secure in place. Use on right or left hand.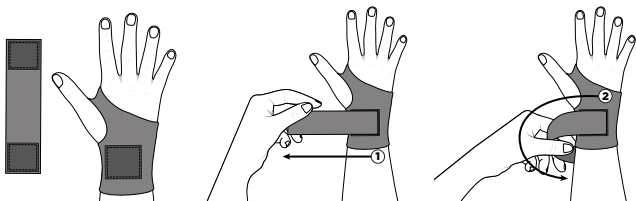


680/685

1. With the elastic loops undone, insert hand into single opening end of support, feeding thumb through the small hole and remaining fingers through the larger opening.
2. Tug at base of wrist until material is even.
3. Wrap elastic strap(s) around wrist until the level of support feels comfortable. Use on right or left hand.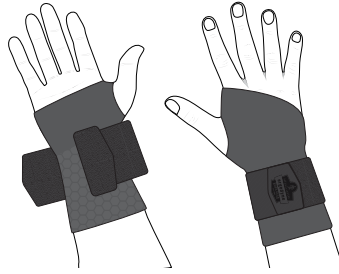


4000/4010

1. Place around wrist (right or left, as stated) aligning the wrist to the orange line indicator.
2. Fasten hook-and-loop closure on back of wrist.
3. Place palm strap between thumb and index finger. Attach end of strap to hook-and-loop closure on back of wrist.
4. Wrap elastic strap across top of the wrist and fasten at desired tension. Right or left hand support must be used on correct hand.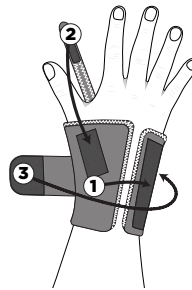


4005/4015

1. With the elastic straps undone, insert hand into single opening end of support, feeding your thumb through the small hole and remaining fingers through the larger opening.
2. Tug at base of wrist until thumb is completely through.
3. Wrap elastic strap(s) around wrist until the level of support feels comfortable. Right or left hand support must be used on correct hand.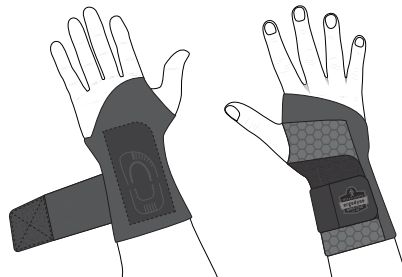

4020

1. Loosen closure.
2. Slide hand (left or right, as stated) through bottom opening of wrist support. Position palm strap between thumb and index finger.
3. Pull closure tab up and toward the inside of the wrist to meet hook-and-loop closure. Fasten at desired tension. Right or left hand support must be used on correct hand.

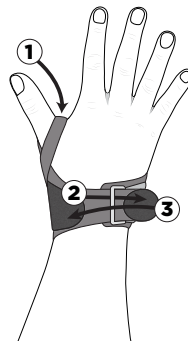CARE INSTRUCTIONS:

Machine wash cold; air dry.

* We cannot guarantee wearing personal protective equipment will prevent injury.

MANUFACTURER

Ergodyne
1021 Bandana Boulevard East, Suite 220
Saint Paul, MN, USA 55108
+1.800.225.8238

AUTHORIZED UK REPRESENTATIVE

Authorised Rep Compliance Ltd.
Arc House, Thurnham
Lancaster, LA2 0DT, UK
Tel. +44 1524 805288

AUTHORIZED EU REPRESENTATIVE

Authorised Rep Compliance Ltd.
Ground Floor, 71 Lower Baggot Street
Dublin, D02 P593, Ireland
Tel. +353 151 33758

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